

# Aqua Aribell PN

Product  
introduction



KMEDICOS CO.,LTD

energize and rejuvenate your skin

# CONTENTS

1.Intro

2.Main ingredients - PN

3.Main ingredients - HA

4.Effect of Aqua Aribell PN

5.Effect of Aqua Aribell PN

6. How to use

7. Quality

# Aribell



# Introduction

energize and rejuvenate your skin



## Aqua Aribell PN

Aqua Aribell PN is a skinbooster solution product also called revitalizant for rejuvenating damaged or stressed skin, restoration and enhancement of skin regeneration

### **PN+HA Solution**

Aqua Aribell is made of only high-quality ingredients. The composition includes PN that maximized the efficacy and an optimized combination with high-quality Hyaluronic Acid to create and synergize the most powerful effect.

# Main Ingredients: PN (Polynucleotide)

-It is a natural donor of substrates for the synthesis of its own DNA molecules, stimulates the synthesis of necessary proteins and increases the reparative potential of the skin.

It is a longer piece of DNA so it stays longer in the skin and has longer effect.

**It is one of the latest developments in cosmetology and have a large evidence-based research.**

-PN is a specific nucleotide extracted and refined from salmon, whose base **composition is most similar to that of the human body.** It has almost no foreign body reaction and has a very stable structure to heat. By activating the skin's natural healing, it improves the skin dermis's environment and restores the skin's structure, which ultimately helps to recover the skin.

**-Furthermore, it thickens the dermis to make the skin healthy, increases skin elasticity, and has an effect in restoring to normal the skin damaged by aging and ultraviolet.**

# WHY PN? (Polynucleotide)

Aqua Aribell PN contains PNs – which also includes PDRN. **PN is a purer and more concentrated form of PDRN with stronger regenerative properties.**

With the active ingredient - PN (polynucleotide), the treatment is able to increase the effectiveness of the dermal fibroblast and increase skin's regeneration by as much as 20%.

PN is an effective substance in regenerating tissue made by cutting salmon DNA in a particular size. As it is a longer piece of DNA than regular PDRN and stays longer in the skin, its effect lasts longer.



# Main Ingredients: HA (Hyaluronic Acid)

***Hyaluronic acid is an essential component of the human body regulating the water balance of tissues and participating in the synthesis of collagen and elastin.***

HA is a substance from polymer molecules that freely fills the space of the extracellular matrix, which contributes to the healthy synthesis of collagen and elastin. HA is considered no less necessary, if not more necessary, than collagen. Over the years, acid synthesis is disrupted. As a result, age-related signs such as dry skin and fine wrinkles appear.

***HA injections helps to retain moisture in the skin layers (one molecule of hyaluronic acid attracts about 700 water molecules).***

***Additionally it stimulates the production of skin's own "hyaluron".***

## **HA ACTIVENESS**

Hyaluronic acid acts on the connective tissue cells - fibroblasts. It stimulates their division, activating the mechanism by which fibroblasts produce their own HA, collagen and elastin, restoring the bonds between cells, contributes to their active regeneration.

***As a result, the skin looks moisturized, supple and smooth, without sagging and premature wrinkles.***

## Main Ingredients: HA (Hyaluronic Acid)

*According to scientific studies, when using hyaluronic acid together with polynucleotide, the effect of polynucleotides on human fibroblasts is enhanced. (An increase in cell growth is observed).*

# Effects of Aqua Aribell PN

1. Recovers damaged cells and stimulates skin repair processes
2. Improves the elasticity of the skin
3. Improve fine wrinkles
4. Narrowing of pores
5. Restores microcirculation in tissues
6. Reduces the process of the appearance of acne and inflammation
7. Facilitates and accelerates the healing of scars
8. Preventing the formation of new wrinkles
9. Increases the metabolic activity



Aribell



# Effects of Aqua Aribell PN

**Effect:** well-moisturized skin, smoothing out small facial wrinkles around the eyes and lips, quick recovery of the skin after a traumatic damage.

**Using:** The product is widely used not only for the face, but also for the hands, wrists, decollete, neck and in places of obvious dehydration.

**Result:** fresh, smooth, firm, elastic skin, moisturized and protected from negative environmental factors. The result is visible after the first step.



# Effects of Aqua Aribell PN



Aribell

# Aqua Aribell PN How to use



1 procedure every 7-12 days 3-4 times for the first time, then 1 supporting procedure per month for 2 months, then every 2-6 months 1 prophylactic procedure.

Aribell

# Aribell products' quality

- Aribell products are manufactured exclusively in a qualified medical products and instruments (not cosmetics) factory, subject to the strictest international standards.
- During the production of products, constant monitoring is carried out to identify even the smallest defects.
- Factory acquired quality related certificates, such as GMP and ISO 13485
- All products have been tested for years for quality and efficiency by professionals of Korean plastic surgery clinics.



Aribell



# Aribell

KMEDICOS CO., LTD

Email:

[sales@kmedicos.com](mailto:sales@kmedicos.com)

Website:

[www.kmedicos.com](http://www.kmedicos.com)